

10 step process: sample collection

10 step process for sample collection using mobile app

1




Sanitise or
wash + dry hands.

2



Select a tube and
a plastic biohazard bag.

3

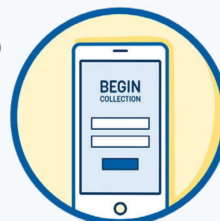

Open the Covid-19
Saliva Testing App.

4



Scan barcode on tube.

5



Remove lid from tube. Begin to
form saliva in your mouth and
allow it to dribble into the tube.

6



Keep dribbling until your
saliva reaches the label
(at least 1ml).

7



Tightly screw lid back onto tube
and re-scan the barcode on tube.

8



Great – nearly done!
Put tube of saliva into plastic
biohazard bag and seal bag.

9



Take bag with tube in it to
the drop off point. Scan the 2D
barcode on the drop off container
and deposit your sample.

10



Sanitise again.
That's it – thank you!



You will get your results
via text message.



For more information visit
www.Covid19SalivaTesting.co.nz.